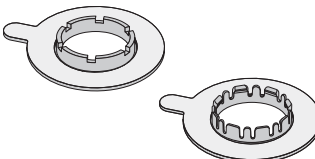
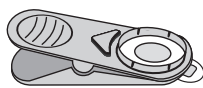
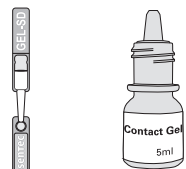
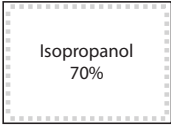
Directions for Lay Users

sentec.

SDM 	EN Sentec Digital Monitor / Docking Station DE Sentec Digital Monitor / Docking Station ES Monitor digital Sentec / Estación de calibración FR Sentec Digital Monitor / Station d'accueil IT Sentec Digital Monitor / Docking Station JA センテックデジタルモニタ / ドッキングステーション NL Sentec Digital Monitor / Docking Station NO Sentec Digital Monitor / Dokkingstasjon PL Monitor cyfrowy SenTec / Stacja dokująca PT Monitor digital Sentec / Docking Station ZH SenTec 数字监控仪 / 校准感应室 
VS-A/P/N 	EN V-Sign™ Sensor 2 ("Sensor") DE V-Sign™ Sensor 2 ES Sensor V-Sign™ 2 ("Sensor") FR Capteur V-Sign™ 2 IT Sensore V-Sign™ 2 JA V-Sign™ センサ 2 (「センサ」) NL V-Sign™ Sensor 2 NO V-Sign™-sensor 2 ("sensor") PL Czujnik 2 V-Sign™ („czujnik") PT Sensor V-Sign™ 2 ("sensor") ZH V-Sign™ 传感器 2(传感器)
MAR- / MARE-MI/5F 	EN Multi-Site Attachment Ring DE Befestigungsring ES Anillo de fijación multizona FR Anneaux de fixation IT Anello adesivo JA マルチサイト・アタッチメントリング NL Bevestigingsring NO Festering PL Pierścień mocujący PT Anel de fixaçao multilocalização ZH 多部位联结环
EC-MI 	EN Ear Clip DE Befestigungsring ES Clip de oreja FR Pince d'oreille IT Clip Auricolare JA ヤークリップ NL Oorclip NO Ørekliips PL Klips nauszny PT Clipse de orelha ZH 耳夹
GEL-SD / GEL-04 	EN Contact Gel DE Kontaktgel ES Gel de contacto FR Gel de contact IT Gel di Contatto JA コンタクトジェル NL Contactgel NO Kontaktgel PL Żel przewodzący PT Clipse de orelha ZH 接触凝胶 

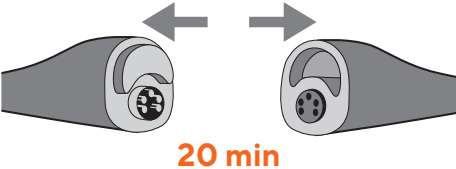
Isopropanol 70%	EN Isopropanol 70% soaked wipe
	DE Tuch mit 70 % Isopropanol
	ES Torunda empapada en isopropanol al 70 %
	FR Lingette imbibée d’isopropanol à 70 %
	IT Salvietta inumidita con isopropanolo al 70%
	JA イソプロパノール70%に浸したワイブ
	NL In 70% isopropanol gedrenkt doekje
	NO Serviett fuktet med isopropanol 70 %
	PL Ściereczka namoczona izopropanolem 70%
	PT Toalhete embebido em isopropanol a 70%
	ZH 70%异丙醇湿巾

EN Directions for Lay Users	4
DE Anleitung für Patienten	8
ES Instrucciones para usuarios principiantes	12
FR Instructions pour les utilisateurs débutants	16
IT Istruzioni per gli utilizzatori comuni	20
JA 一般使用者向け使用説明書	24
NL Instructies voor niet-professionele gebruikers	28
NO Instrukser for brukere	32
PL Instrukcja dla użytkowników bez wykształcenia medycznego	36
PT Instruções para utilizadores leigos	40
ZH 非专业用户使用指南	44

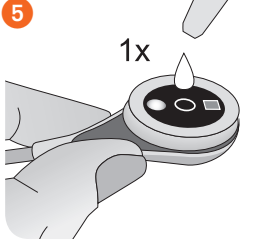
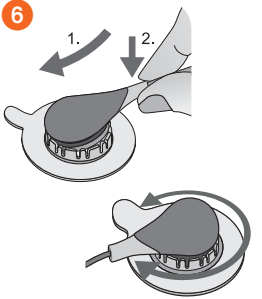
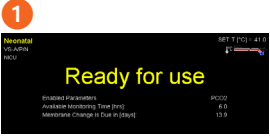
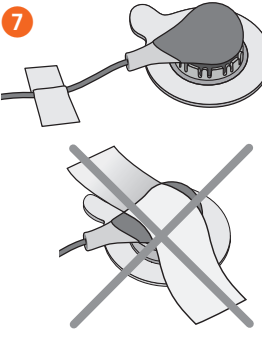
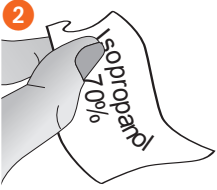
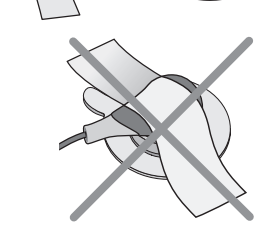
EN | Directions for Lay Users

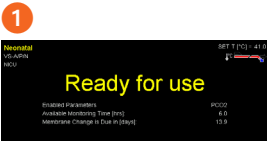
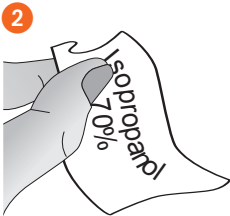
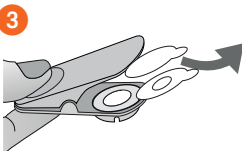
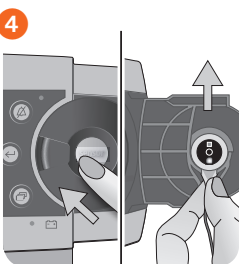
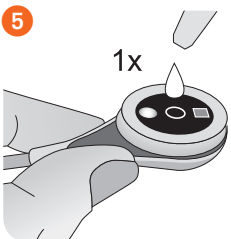
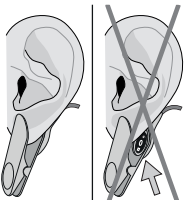
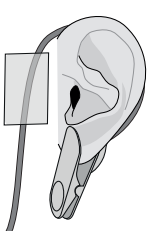
Important

The device may only be installed by a home care provider’s instructed person.
Do not make any changes to the device settings.
Do not make any changes to the mains plug of the device and secure the mains cable against squeezing or bending.
In the event of unforeseen incidents, error messages, unexplained changes in the performance of the device, or if you require any other assistance, please contact your home care provider’s instructed person. If in doubt, you can interrupt the measurement by removing the sensor, cleaning it and placing it in the docking station.
If you need to interrupt a session after monitoring has started (e.g. to go to the bathroom), you may disconnect the sensor for up to 20 minutes. If you disconnect the sensor for more than 30 minutes, the recording will be lost.  20 min
Keep electronic devices and cables out of reach of children and pets.
Make sure that children do not swallow Contact Gel or small parts such as the sensor, the Multi-Site Attachment Rings or Ear Clip.
Only use Sentec Contact Gel. Do not use any other contact gel/liquid.
Do not pour any liquid on the SDM or its accessories. If the SDM has been wetted accidentally, it must be removed from AC power, wiped dry externally, allowed to dry thoroughly, and inspected by qualified service personnel before further use. The SDM is protected against vertically falling drops of water, the Isolation Transformer is protected against splash water.
Keep the device away from heat sources (e.g. heating) and at least 50 cm from flammable materials.
Mobile telephones and other radio communication devices must be kept 1 m away from the device.
Do not cover the device.
Do not use a non-invasive blood pressure cuff or other constricting devices on the same limb as the sensor. This may result in inaccurate measurements.
Protect all devices from lint, dust and direct sunlight.
High ambient light conditions may result in inaccurate measurements.

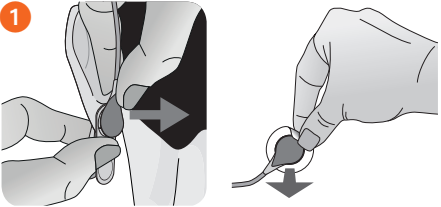
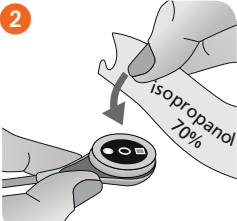
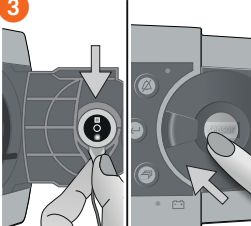
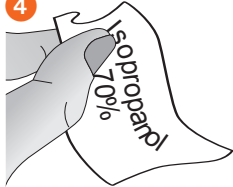
A) Sensor application using MAR/e-MI or MAR/e-SF

	https://youtu.be/g3aQbOqge5Y		Apply one drop of Contact Gel to the sensor surface Note: Avoid contact with injured skin. Do not use when exhibiting allergic reactions to one of the components.
	Measurement Sites		Click sensor into MAR/e and rotate sensor to spread Contact Gel.
	Ensure SDM displays "Ready for use".		Secure cable with an adhesive tape. Important Note: Route the sensor cable properly (e.g. with a Clothing Clip) to avoid entanglement or strangulation.
	Clean Measurement Site selected by the home care provider, using Isopropanol 70% soaked wipe.		Pull adhesive tape off MAR/e and attach MAR/e to measurement site, pressing gently around the ring.
	Open Docking Station, remove sensor, close Docking Station. Note: Do not touch the sensor surface with any hard objects or fingernails. Otherwise, the sensor could be damaged.		Open Docking Station, remove sensor, close Docking Station. Note: Do not touch the sensor surface with any hard objects or fingernails. Otherwise, the sensor could be damaged.

B) Sensor application using EC-MI

	https://youtu.be/vBeiLWBHmPk		Measurement Sites
	Ensure SDM displays "Ready for use".		
	Clean Measurement Site selected by the home care provider, using Isopropanol 70% soaked wipe.		
	Pull both adhesive tapes off Ear Clip, attach Ear Clip to earlobe with retainer ring on the back side.		
	Open Docking Station, remove sensor, close Docking Station. Note: Do not touch the sensor surface with any hard objects or fingernails. Otherwise, the sensor could be damaged.		
	Apply one drop of Contact Gel to the sensor surface Note: Avoid contact with injured skin. Do not use when exhibiting allergic reactions to one of the components.		
	Click sensor into Ear Clip rotate sensor to spread Contact Gel Note: Earlobe must cover the entire sensor surface. Ensure to hold the sensor such that the contact liquid does not run off the sensor face.		
	Wrap the sensor cable around the ear and secure it with an adhesive tape. Note: Do not lie on Ear Clip as this may lead to incorrect measurements. Important Note: Route the sensor cable properly (e.g. with a Clothing Clip) to avoid entanglement or strangulation.		

C) Sensor removal

1 	When monitoring is completed, take off Ear Clip or gently peel off Multi-Site Attachment Ring, then remove sensor from retainer ring
2 	Clean sensor and cable with Isopropanol 70% soaked wipe.
3 	Open Docking Station, put sensor into cradle inside the door with the red light visible, close Docking Station. Note: Do not turn off the device.
4 	Clean measurement site with Isopropanol 70% soaked wipe. Note: Your home care provider will ensure professional and hygienic maintenance of the system once returned.

Important

Do not turn off the device at the end of the measurement.

The evaluation and analysis of the measurements is carried out by the home care provider. Analysis of data is done by the supervising medical doctor.

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Sentec AG